

cardioCream (Paste for ECG)

Z-101BC

General

The Z-101BC cardioCream is cream designed for attaching Nihon Kohden electrodes in ECG measurement. Refer to the manual for the equipment you will use with this. Follow the physician's instructions when using this cream.

Safety Information

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

- Use the cream only for ECG measurement.
- If rash, redness, itch, swelling or abnormality appears on skin, wash the cream off immediately and do not use it.
- Do not use the cream on a wound or inflamed skin.
- Do not use the cream past its stated lifetime.
- If the cream gets in the eyes or mouth, immediately wash it out with plenty of water and consult an eye doctor or a physician.
- If the cream is swallowed, consult a physician.

⚠ CAUTION

Do not allow any cream to remain on the electrodes and lead tips because this may cause the electrodes and lead tips to rust.

⚠ CAUTION

- Store the cream in a cool and dry place. Avoid direct sunlight and high temperatures above 40°C (104°F). If stored at temperatures above 40°C (104°F), the cream may separate.
- Tighten the cap to prevent the cream from drying. If the cream is dried, the ECG waveform cannot be recorded.
- Keep the cream out of the patient's reach.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 24 Apr 2019
2nd Edition: 14 Dec 2022

