

# skinPure

## YZ-0019

### General

This is a skin preparation gel that helps acquire a stable waveform during vital sign monitoring.

### Safety Information

**⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

#### **⚠ CAUTION**

- Do not use "skinPure" for neonate or low birth weight infant.
- Do not use "skinPure" for any other purpose.
- Do not use "skinPure" on hypersensitive, injured or inflamed skin.
- Do not rub skin with "skinPure" too hard. It will inflame the skin.
- If rash, itch, swelling or abnormality appears on skin, wash "skinPure" off immediately and do not use it.
- If "skinPure" gets in the eyes or mouth, immediately wash it out with plenty of water and consult an eye doctor or a physician.
- If "skinPure" is swallowed, consult a physician.
- Follow the physician's instructions to use "skinPure".
- Do not use "skinPure" past its stated lifetime.
- Keep "skinPure" out of the patient's reach.
- Store "skinPure" in a cool place away from direct sunlight.
- Store "skinPure" with the cap tight. If "skinPure" is left uncapped, it dries up and cannot reduce contact impedance.
- Follow your local laws for disposing of "skinPure".
- The contents may partly solidify but it doesn't affect use of the "skinPure".
- "skinPure" contains sea sand and may contain black particles. This does not affect use.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>

**EC REP** European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffenstrasse 10, 61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 11 Apr 2019  
2nd Edition: 28 Apr 2023

