

Battery Charger

SB-101V

General

The SB-101V battery charger is for charging the SB-121V lithium ion battery.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Never use the battery charger in the presence of any flammable anesthetic gas or high concentration oxygen atmosphere. Failure to follow this warning may cause explosion or fire.

⚠ WARNING

Never use the battery charger in a hyperbaric oxygen chamber. Failure to follow this warning may cause explosion or fire.

⚠ WARNING

Never use a battery pack which is damaged, discolored or has leakage. It may cause overheating, explosion or fire.

⚠ WARNING

Only use the provided AC adapter and specified optional power cord and connect the power cord to a 3-pin AC outlet which is properly grounded. Otherwise, it may result in electrical shock or injury to the operator.

⚠ WARNING

Do not disassemble or modify the battery charger. It might cause skin burn, fire, electrical shock or injury.

⚠ CAUTION

If the battery charger seems to be faulty, immediately disconnect the power cord from the AC outlet, attach a "Do not use" or "To be repaired" label on the battery charger and contact your Nihon Kohden representative.

⚠ CAUTION

Never disassemble or repair the battery charger. If there is any problem with the battery charger, contact your Nihon Kohden representative.

⚠ CAUTION

Only charge a Nihon Kohden specified battery pack. If an unspecified battery pack is charged, it may cause explosion.

⚠ CAUTION

Refer to the manuals of the battery pack and the defibrillator on which the battery pack is used for details on handling the battery pack.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

⚠ CAUTION

Before maintenance, cleaning or disinfection, disconnect the power cord plug from the AC outlet. Failure to follow this instruction may result in electrical shock and battery charger malfunction.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 02 Apr 2019
2nd Edition: 06 Jul 2023

